

DULAS VC225ILR
Mains Powered Vaccine Refrigerator
User Manual




Table of Contents

1. COMPONENTS CHECKLIST	3
2. SAFETY INSTRUCTIONS	4
3. YOUR DULAS MAINS POWERED REFRIGERATOR.....	6
4. REFRIGERATOR COMPONENTS.....	7
5. MATERIAL SAFETY INFORMATION	8
6. END OF LIFE & PRODUCT RECYCLING	9
7. REFRIGERATOR INSTALLATION	11
8. REFRIGERATOR LOADING	12
9. STATUS LIGHT	13
10. USER TRAINING.....	14
11. MAINTENANCE TASKS.....	15
12. VC225ILR SYSTEM LAYOUT	15

1. Components Checklist

DESCRIPTION	QUANTITY
REFRIGERATOR	
VC225ILR Refrigerator Cabinet	1
Refrigerator Baskets	6
Refrigerator Key	2
Other	
Power Supply PS-ILR-001	1
Screws (5x40mm) – for wall mount	5
Wall plugs (Brown) – for wall mount	5
Screws	10
Wall plugs (Red)	10
Cable ties	10
Cable tie bases	10
MC connection tool	1


2. Safety Instructions

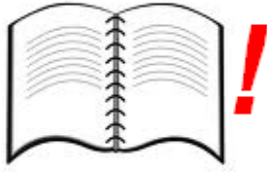

Read the Manual - please read all instructions completely before installing. Installation and system servicing should only be carried out by a qualified and competent engineer. Work performed by persons with insufficient technical knowledge may adversely affect the performance of the unit or cause physical injury or damage to the equipment.



The refrigerator is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction. Children must be supervised not to play with the appliance. This refrigerator is for professional use only. It is not designed for household use.

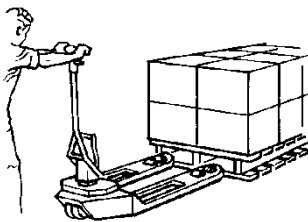


Refrigerator and power system are heavy - please observe good manual handling procedures when moving the refrigerator and solar power system. Only use suitable transport equipment.



Live electrical components – The refrigerator operates from low voltage DC. There are high voltage live circuits within the accompanying power supply which should only be accessed by qualified technicians.



Electrical danger – under fault conditions high electrical currents can occur. Never bypass a circuit breaker and always replace with the same type and rating.

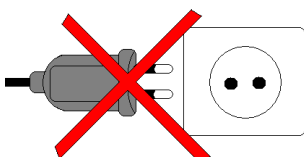

DC power only - this refrigerator is designed for operation with the supplied power system only. Do not connect the refrigerator to other power supplies.




Dangerous substances - this refrigerator contains substances that are not biodegradable and can cause harm, please ensure that all components of the refrigerator are disposed of correctly, see page 8 for more information. Do not damage the internal side panels of the compartment.




Keep Dry – This refrigerator is for indoor use only. Do not expose it to rain.

WARNING: This unit's cooling system contains flammable refrigerant. Due to this fact, the following information is particularly important:

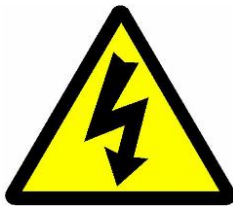

Warning: Do not damage the refrigeration circuit. Ensure that sharp or pointed objects do not come into contact with the refrigeration circuit.

Warning: Do not use mechanical devices or other means to accelerate the defrosting process, other than those recommended by the manufacturer.

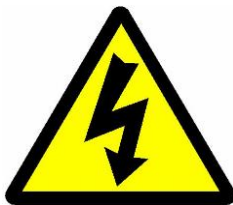

Warning: Keep ventilation openings, in the appliance enclosure or in the built-in structure, clear of obstruction.

Warning: Do not use electrical equipment inside the appliance.

Warning: Do not store explosive substances such as aerosol cans with a flammable propellant inside this appliance.

3. Your Dulas Mains Powered Refrigerator.

Congratulations on choosing the Dulas VC225ILR vaccine refrigerator. With over 30 years of engineering experience in the production of vaccine refrigerators, Dulas is considered one of the market leaders in this field.

The VC225ILR is Dulas' latest innovation in vaccine refrigeration. Using technology adapted from our ground breaking Solar Direct Drive refrigerators we have created a mains powered refrigerator that is ideally designed for intermittent electricity supplies. With advanced insulation and a robust phase change lining, our ILR delivers an exceptionally long holdover while also providing 100% freeze free protection for your vaccines. This new technology offers the same renowned Dulas quality and reliability that have always kept your vaccines safe, but in a simpler and more efficient package.

How does the Dulas ILR work?

When mains power is available the refrigerator runs, cooling the vaccine compartment and its contents. In many situations electricity is intermittent, causing problems for traditional mains powered refrigerators. To maintain the temperature when the power is off the Dulas ILR has a phase change lining. This lining acts as a "cold store" and stores energy that keeps the temperature inside the refrigerator stable when electricity is not available. The temperature of the cold store in the Dulas ILR is naturally between +4 to +5°C which is the perfect temperature required for safe vaccine storage and means that the product is 100% Freeze Free.

In order to store sufficient energy to deliver the specified holdovers, the refrigerator must be allowed time to charge its cold store. Refer to the table below to see how long it will take for the Dulas ILR to be ready to store vaccines:

Total No. of hours the power is off each day	Running time required before full holdover is achieved (at +43°C ambient)	Continuous operation possible once cool down has been achieved
0	2 days	Y
12	4.5 days	Y
16	7 days	Y
20	14 days	Y



This table applies to a new refrigerator or whenever the internal temperature has gone above +8°C

Recommended power supply

The Dulas ILR must be connected to the supplied Dulas AC power supply (PS-ILR-001). This power supply delivers the DC electricity required by the refrigerator. Connect the PS-ILR-001 power supply to an AC input of:

- Nominal 230V 50Hz single phase supply rated to at least 5A. (110V and or 60Hz models available)
- Voltage range: 160 – 300V; Frequency range: 45 – 65 Hz

4. Refrigerator Components

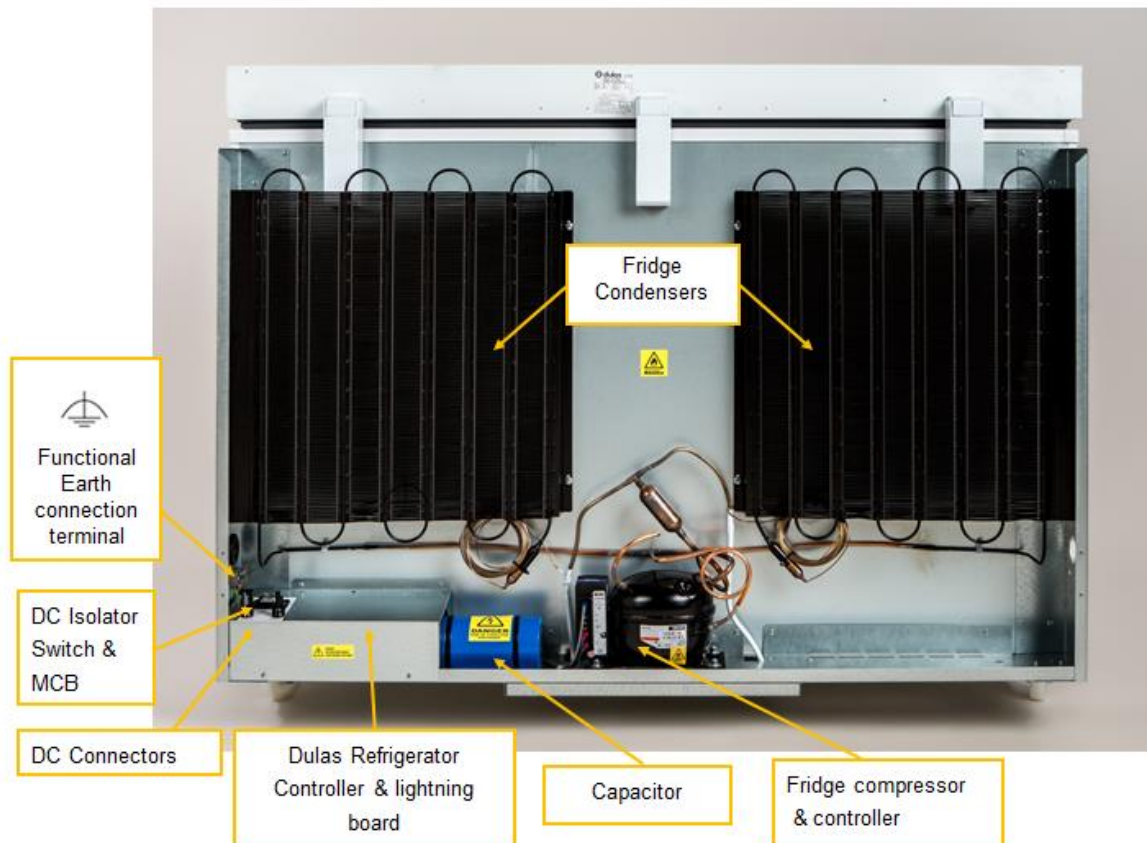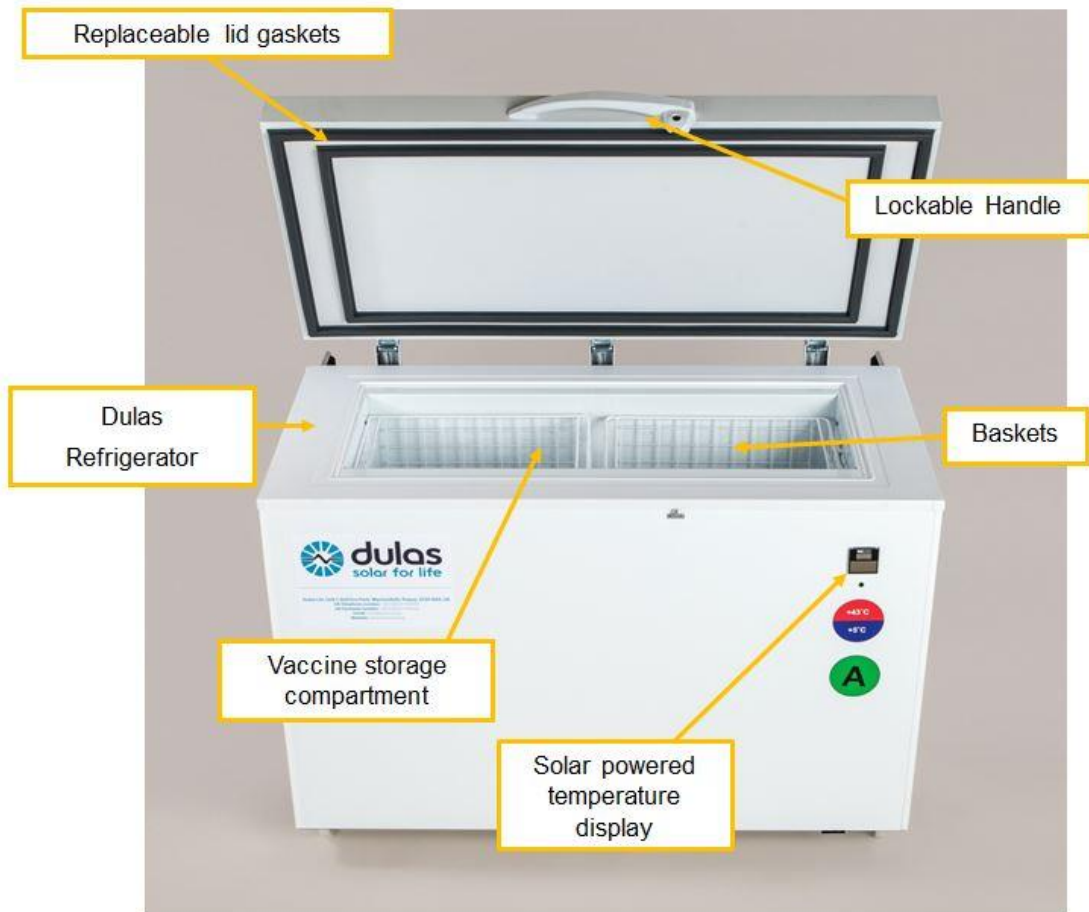

5. Material Safety Information

THE REFRIGERATOR

The Dulas VC2225ILR conforms to the European directive 2002/95/EC on hazardous substances and does not contain the following elements: lead, cadmium, mercury, hexavalent chrome, PBB or PBDE.

The absence of ozone destroying materials is in accordance with EC 1005/2009 (CFC-free)

It is compliant with European Standards (CE) EN60335-1:2010 & EN60335-2-24:2010.

R600a refrigerant – Isobutane, is highly flammable when mixed with air. Do not inhale, ingest, and avoid contact to skin. Ensure that any work carried out on the refrigeration circuit is done by a competent refrigeration engineer.

THE PHASE CHANGE MATERIAL (PCM)

The refrigerator contains an organic phase change material (PCM) that provides the energy store for the refrigerator. This PCM material is a paraffin wax that is liquid at room temperature and is stored in large tanks secured inside the walls of the refrigerator. In normal use the user will never be aware of its presence. This PCM is classified under regulation (EC) No 1272/2008 as a category 1 aspiration hazard and is classified under regulation 67/548/EWG, 1999/45/EG as having the potential to cause physical harm (R65 & R66). At normal ambient temperatures this product will be unlikely to present an inhalation hazard because of its low volatility. At high temperature aerosol/mist can cause an irritation of the respiratory tract.



H304 May be fatal if swallowed and enters airways.

P301 + P310 IF SWALLOWED: Immediately call a POISON CENTRE or physician.



P331 DO NOT induce vomiting.

R65: Harmful: may cause lung damage if swallowed.

R65: Repeated exposure may cause skin dryness or cracking.

In the event of a PCM spillage:

- Wear suitable protective clothing, gloves and safety glasses when clearing up spillages.
- Use an absorbent substance such as sand to contain any escaping material.
- If the PCM contaminates rivers and lakes or sewers inform authorities. Avoid water ingress underground. Do not flush into surface water or sanitary sewer system.

6. End of Life & Product Recycling

PACKAGING

The packaging is made from recyclable / reusable materials and should be brought to an official recycling centre. The materials used are:

- Heat treated (HT) timber, certified by UK Forestry Commission (ISPM No. 15 Mark)
- Corrugated cardboard / Cardboard
- Polyethylene plastic
- Moulded polystyrene parts
- Steel reinforcing straps

WARNING – Keep packaging away from children - danger of suffocation from plastic!

WORN OUT REFRIGERATOR

The refrigerator still contains valuable materials and is not to be disposed of in the normal household waste.

- Ensure that the cooling circuit of the worn out device is not damaged when being transported.
- Information on the coolant used is on the type plate on the back of the refrigerator.
- The walls of the refrigerator contain an organic phase change material that can be harmful and must be disposed of safely. Please contact Dulas for advice.
- Worn out devices must be professionally disposed of in accordance with local regulations and legislation.

Potentially hazardous components / materials are:

- Refrigerant gas (R600a)
- Phase change material (PCM) - paraffins, normal C5-20
- Electrolytic capacitor
- Electronic components – Refrigerator controller
Compressor controller
Refrigerator compartment temperature display

Disposal of Phase Change Material (PCM)

The PCM used in the Dulas refrigerator has a very long life and does not degrade through use. At the end of the refrigerators useful life the PCM should be either recycled or disposed of in accordance with applicable regional, national and local laws and regulations. Contact your local waste disposal authority for advice. The product can be incinerated in accordance with local regulations. In the EU it has a Waste Disposal Code (EWC): *13 08 99 oil waste not otherwise specified*.

REFRIGERATOR POWER SUPPLY

Do not dispose of this equipment with normal domestic waste!

To comply with the European Directive 2002/96/EC on Waste Electrical and Electronic Equipment and its implementation as national law, electrical equipment that has reached the end of its life must be collected separately and returned to an approved recycling facility. Any device that you no longer require must be returned to our agent, or find out about the approved collection and recycling facilities in your area.

Ignoring this European Directive may have potentially adverse effects on the environment and your health!

7. Refrigerator Installation

DO NOT position the refrigerator in direct sunlight or next to any heat sources. Make sure the refrigerator is positioned to provide good ventilation and ease of access for maintenance. Ventilation is particularly important for the condensers and compressors.

- 1) Position the refrigerator on a level surface; wipe all internal and external surfaces with a clean cloth.



- 2) Check the DC isolator at the back of the cabinet is in the '0' position.
- 3) Connect the refrigerator to the Dulas PS-ILR-001 power supply. Check that there is sufficient slack in the cable to allow access to the back of the refrigerator.
- 4) Plug the Dulas PS-ILR-001 into a suitable AC outlet and turn it on. Then switch the DC isolator to the "1" position. Within 1 minute the compressor will start if there is AC power. You will be able to hear the compressor operating and feel the condensers getting warm.

- 5) Check that the refrigerator compartment temperature display on the front of the cabinet is operating. If there is insufficient natural light shine a torch on the display for a few seconds.



- 6) Ensure that the lid is firmly closed.
- 7) Check that all cables are securely supported so that they will not be pulled loose if the refrigerator is moved for cleaning.
- 8) Roll the refrigerator into its final position and apply the brakes on the castors to secure it.

8. Refrigerator Loading

IN ORDER TO KEEP INTERNAL TEMPERATURES STABLE AND MINIMISE ENERGY CONSUMPTION, ONLY OPEN THE LID WHEN STRICTLY NECESSARY.

The VC225ILR refrigerator has a single compartment specifically for the storage of vaccines, see Section 4. This compartment is designed to maintain a stable temperature between +2°C and +8°C across an ambient temperature range from +5°C to +43°C. There are no points within the vaccine compartment that drop below freezing and thus no danger of freezing vaccines.

Optional stacking baskets are available. These are designed to make stock management easier and encourage air circulation within the compartment.

Try to keep the lid open for as short a time as possible.

Stock management

- Allocate a certain space for each different type of vaccine, so that they may be located rapidly.
- Check the vaccine labels and adhere to any special requirements.
- Always use before the expiry date (store older vaccines on top).
- Store the most frequently used vaccines in the top baskets for easy access.

WARNING: ALWAYS FOLLOW THESE VACCINE STORAGE GUIDELINES

9. Status Light

The green LED light on the front of the refrigerator illuminates to indicate that the compressor is operating. The compressor will run if the refrigerator temperature is above about 4 degrees and there is sufficient solar power.

Normal operation

The refrigerator warms up slightly overnight. It is usual for the compressor to start running shortly after sunrise. It will run continuously for a few hours to cool the refrigerator. After this time the compressor will begin cycling on and off to maintain the correct internal temperature. This will continue until the sun sets at the end of the day. At night there is no power to the refrigerator and so the compressor will not run nor will the LED light operate.

10. User Training

The end user of the VC225ILR refrigerator should be instructed in the simple processes outlined below and also those in “Section 11 Maintenance tasks”. It should be stressed that these tasks are essential to ensure correct and reliable operation of the refrigerator.

Caution: failure to follow these instructions may lead to system failure and vaccine wastage.

If you have a problem, please contact your appointed engineer.

Turning on

When turning on for the first time or when the refrigerator has been turned off for more than 3 days:

- Turn the power on and allow the refrigerator to run in accordance with the table below:

Total No. of hours the power is off each day	Running time required before full holdover is achieved (at +43°C ambient)	Continuous operation possible once cool down has been achieved
0	2 days	Y
12	4.5 days	Y
16	7 days	Y
20	14 days	Y

- Before loading with vaccines, ensure that the temperature display on the front of the refrigerator reads +4.5°C.

Daily Routine

- Keep the lid closed, open only when required and lock after use.
- Store the most frequently used vaccines in the top baskets for easy access.
- Do not keep food or drink in the refrigerator.

Vaccine Storage:

- Only store vaccines in the refrigerator.
- Always store vaccine in its original packaging.
- Always keep the same type of vaccine together and store your vaccine neatly.
- Always use old vaccines before new ones; before use make sure the vaccine has not expired.

11. Maintenance Tasks

Every morning and afternoon:

- Check the temperature
- Fill in the daily record sheet

On the first day of every month:

- Clean the refrigerator, condensers and compressors

Every 6 months:

- Check all mechanical fixings and electrical connections.

12. VC225ILR System Layout

